EN Integration of restorative procedures in the dental office

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20-year old male patient.

COMPLAINT

Dissatisfaction with the shape and shade of the teeth.

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EN CLINICAL/RADIOGRAPHIC EXAM

Tooth 21 had endodontic treatment, however it showed darkening due to a previous trauma. The exam also showed the misalignment of the gingival zeniths of the superior anterior teeth. In those elements, there was also wear and small fractures.

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TREATMENT

For this patient, a periodontal plastic surgery was initially planned and executed. Later, internal whitening of tooth 21 was performed followed by the placement of a fiberglass post and direct veneer with Opallis. Then, external whitening from second pre-molar to pre-molar was carried out followed by direct restoration to improve the aesthetic composition of anterior-superior teeth.

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// To integrate restorations to the smile in a natural way, it is necessary to make use of a multidisciplinary approach. FGM provides quality materials for us to accomplish excellent rehabilitations. //

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Prof. Dr. Maciel Júnior

STEP BY STEP | PASO A PASO

1. Initial clinical situation showing great gingival exposure, generalized darkening and wear.
2. Intraoral view .
3. After cervical sealing, internal whitening with Whiteness Perborato, was carried out for 5 sessions. In each session, teeth were sealed with Maxxion C glass ionomer.
4. External whitening was performed according to the combined technique with Whiteness HP Blue 35% and 4 syringes of Whiteness Perfect 10%.

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5. Fiberglass post Whitepost DC being tested before preparation and cementation in tooth 21.
6. Situation after whitening and before restorative procedures.
7. Application of Ambar adhesive.
8. Placement of Opallis T-Neutral on the palate barrier.
9. Restorations executed with D-Bleach, E-Bleach M y H, and VH.

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10. Palatal shell with T-Neutral to begin the extratification of the veneer on teeth 21.

11. Application of a layer of shades OP and D-Bleach.

12. Application of T-Blue and OP in the incisal area.

13. Last layers with composite for whitened enamel E-Bleach H and M, and VH.

14. Finishing and polishing with abrasive disks Diamond Pro.

15. Final polishing with Diamond Flex disk and Diamond Excel paste.

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16. Result of the restoration after finishing and polishing.

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18

19

18 and 19. Initial and final smile of the patient.

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FGM MATERIALS USED | MATERIALES FGM UTILIZADOS

WHITENING GEL FOR NON-VITAL TEETH
(CARBAMIDE PEROXIDE)

Whiteness Perborato

IN-OFFICE WHITENING GEL
(HYDROGEN PEROXIDE)

Whiteness HP Blue 35%

TAKE-HOME WHITENING GEL
(CARBAMIDE PEROXIDE)

Whiteness Perfect 10%

LIGHT-CURING NANOHYDRID COMPOSITE
Opallis

7th GENERATION LIGHT-CURING ADHESIVE
Ambar Universal

FIBERGLASS POST
Whitepost DC

GLASS IONOMER CEMENT
Maxxion C

FELT DISKS
Diamond Flex

SANDPAPER DISKS
Diamond Pro

DIAMOND POLISHING PASTE
Diamond Excel

DISPOSABLE MICROAPPLICATOR
Cavibrush

LIGHT-CURING GINGIVAL BARRIER
Top dam

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