EN Whitepost associated with a multifunction resin cement to simplify dental rehabilitation

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QUEJA

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EN CLINICAL/RADIOGRAPHIC EXAM

During the exam, a ceramic veneer with color alteration was observed on tooth 11 as well as cervical and incisal unevenness when compared to tooth 21. Radiographically, tooth 11 had already received a satisfactory endodontic treatment.

TREATMENT

The treatment was initiated with the increase of the clinical crown of tooth 11 to align the margin and the gingival zenith. After tissue maturation, the pre-existing restoration was removed and a fiberglass intraradicular retainer (Whitepost DC) was installed by using the direct technique. After cementation of the post and core build up with Allcem Core, a ceramic total crown was installed.

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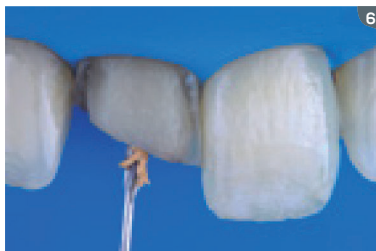
7 to 9. Conduct preparation with the drill available in the system (Whitepost DC #1), providing an anatomic preparation equivalent to the post format.

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Whitepost

APPROVED BY THE MOST EXIGENT PROFESSIONALS

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DOUBLE TAPER DOBLE CONICIDAD

White Post fits better into the conduit with no need for excessive wear of the conduit walls.

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Whitepost

HIGHLY TRANSLUCENT EXCELENTE TRANSLUCIDEZ

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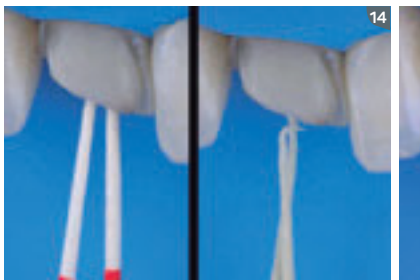
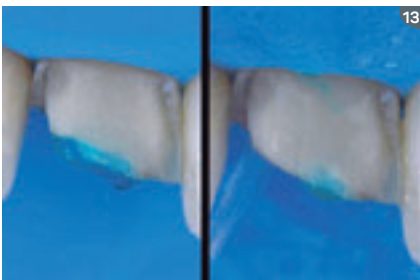
13. Application of 37% phosphoric acid (Condac 37) on the crown enamel for 30 seconds followed by thoroughly washing.

14. Drying with absorbing paper.
14. Secado con papel absorbente.

15. Ambar was actively applied with Cavibrush on the crown portion as well as intra-conduct with light air jets to evaporate the solvent and intra-conduct excess removal with absorbing paper.

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17. The post was treated with alcohol previously to silane (Prosil) application on the surface, waiting for 60 seconds for the solvent to evaporate.

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19. After the adhesive preparation, insertion of the cement into the conduct with the aid of an applicator tip.

20. Post insertion.
20. Inserción del poste.

21. Light-curing for 60 seconds, after the addition of the material for the core build up.

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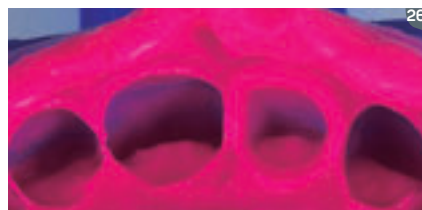
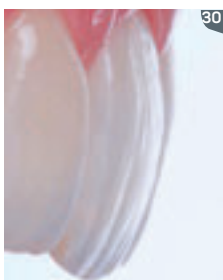
29. In the piece interior, acid etching is performed for 20 seconds with Condac Porcelana 5% and, after that, washing and drying are carried out before silane application (Prosil). Once the adhesive preparation is done, the dual-curing resin cement (Allcem Core) is applied in the interior of the piece which is positioned on the tooth.
29. En el interior de la pieza, se hace el grabado ácido por 20 segundos con Condac Porcelana 5%, lavado y secado para entonces aplicar el silano (Prosil). Realizado el preparo adhesivo, el cemento resinoso de curado doble (Allcem Core) se pone en el interior de la pieza y ésta se posiciona en el diente.

30 and 31. After the excess removal, additional light-curing on the margins and finishing on the cervical terminal, one can notice a well-adapted crown and an excellent result.

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FIBERGLASS POST
Whitepost

POSTE DE FIBRA DE VIDRIO
Whitepost

5th GENERATION LIGHT-CURING ADHESIVE
Ambar

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DUAL-CURING RESIN CEMENT
Allcem Core

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HYDROFLUORIC ACID AT 5%
Condac Porcelana 5%

ÁCIDO HIDROFLUORÍDRICO AL 5%
Condac Porcelana 5%

SILANE
Prosil

SILANO
Prosil